

ORALMEDIC®

Eliminates mouth ulcer pain in seconds!

Uses: One application of **ORALMEDIC®** stops the pain and discomfort of a mouth ulcer in seconds. It works by forming a protective barrier over the ulcer to provide optimum conditions for natural healing.

Instructions:

1. Dry affected area using the cotton drying bud.
2. Hold the applicator with coloured ring up. Gently bend the coloured ring to one side until it snaps. **ORALMEDIC®** will flow to the opposite end.
3. Apply to the affected area for 5-10 seconds, ensuring the entire ulcer is covered. A brief "stinging" sensation is experienced immediately upon application.
4. Rinse mouth thoroughly with water and spit out.



Note: A single application per ulcer treatment is usually sufficient. Ulcer pain remaining immediately after rinsing with water is a sign that part of the ulcer was not covered. If this occurs, repeat application using a new applicator and re-rinse. Do not treat an individual ulcer in more than one treatment session. If pain persists, consult your doctor.

Avoid contact with clothing as it may discolour or damage.

Important Safety information: Do not use if allergic to materials that contain sulphur. Do not use if you are pregnant, or on children under the age of 12. Avoid eye contact. If eye exposure occurs, immediately remove any contact lenses, irrigate eyes for at least 15 minutes with water and seek immediate medical advice.

ORALMEDIC® is for local use on the mucosa in the mouth only. May cause local irritation when applied. If excess irritation occurs during use, rinse thoroughly with plenty of water. If condition persists, consult your doctor or dentist.

Ingredients: HYBENX® (Hydroxybenzenesulfonic Acid, Hydromethoxybenzenesulfonic Acid, Sulfuric Acid, Water, FD&C, Red 40.

How to store this product: Store at room temperature. Maximum 25°C. Keep out of reach and sight of children.



© EPIEN Medical, Inc 2007, St Paul, MN55110, USA.
www.epien.com

HYBENX® is a registered trademark of EPIEN Medical, Inc 2007 ©.

Distributed in the UK by: Ceuta Healthcare Ltd, Hill House,
41 Richmond Hill, Bournemouth BH2 6HS. Tel: 01202 780558
www.ceutahealthcare.com

Distributed in Republic of Ireland by: Kelkin Ltd, 1 Crosslands
Industrial Park, Ballymount Cross, Dublin 12. Tel: 01 4600 400
www.kelkin.ie

EC/REP Interface International Consultancy Ltd,
York, YO42 1YQ.

Text prepared 02/07.

